



WIMBA[®]
PRO

Medical Indications

Indications are suggested use only, WIMBA defers to prescribing clinician for clinical decisions.



WIMBA Pro Indications:

- **1 - Protective Support for Digit Conditions** - deformations, malformations, missing digits, OA in digits - joints and ligaments, varus/valgus instability of digits
- **2- Varius Metacarpal/Metatarsal Conditions**
- **3 - Varus/Valgus Instability of the Joint** - mild to moderate cases. Severe varus/valgus instability will be assessed on case by case basis and some of them might be declined



NOTE: Each varus/valgus instability case is prone to cause contact between skin & orthosis. The limb will be falling to the side and resting on the hinges of the orthosis. For that reason, there will be soft TPU padding on the tokens (hinges), but due to providing appropriate support to the limb, contact cannot be fully prevented.



Suitable Cases:



WIMBA Pro Indications:

- **4 - Protective Support for Mild to Moderate Neurological Damage -** knuckling, weak corrective reflex etc.



NOTE: It should be considered on a case-by-case basis whether a paw segment is necessary/beneficial for the patient. Every effort is made to decrease the necessary difference in height the paw segment will create while maintaining the structural and functional integrity of the orthosis. However, Please note that adding a paw segment will impact the dog's proprioception and will make the limbs length uneven. This is an expected side-effect that can be accepted if paw segment will be beneficial for the treatment, but that should be considered if the dog will do equally with or without the foot component.



- **5 - Post-op/Alternative to Arthrodesis/Failed Arthrodesis**
- **6 - Many cases that were determined too severe for WIMBA Go but can be supported with orthosis**



WIMBA Pro - No Go:

✘ Unsuitable Cases:

- **1 - Severe varus/valgus instability of the joint**
- **2 - Angular bone deformations**
- **3 - Severe neurological cases** - e.g. nerve paralysis, complete bathyanaesthesia, cases in which amputation+prosthesis would be a better choice
- **4 - Small, mini and toy breeds** - it is possible this product will be developed further to cater for all sizes of dogs, but as of now, there is no mini size product.
- **5 - Dogs heavier than 45 kg** - individual consultation with WIMBA team before placing an order; limitations in choice and availability
- **6 - Species other than dogs**

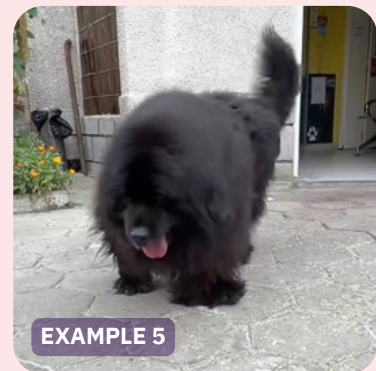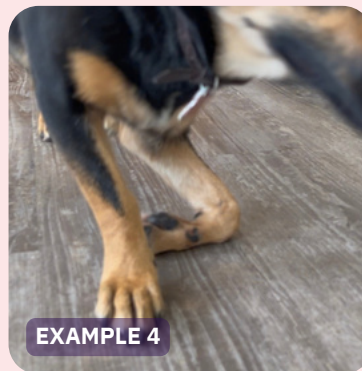

How to Order WIMBA Pro?

Simply download WimbaAPP available for iOS and Android devices.


To order WIMBA Pro devices, you must become a WIMBA Provider Pro.

As a WIMBA Provider, you'll have access to even more advanced solutions designed for complex cases, ensuring your patients get the highest level of orthopedic care. You will receive WIMBA Box with WimbaSCAN equipment needed to order the device.

[Fill in the form here to get started.](#)



Need Help?

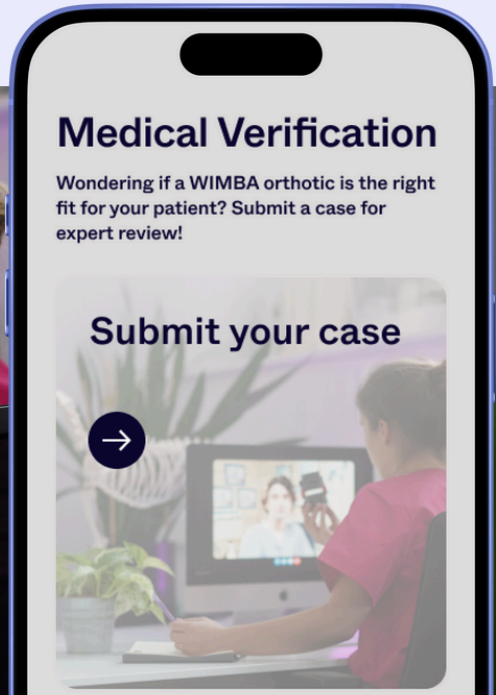

WIMBA: WE ARE YOUR ORTHOTIC CONSULTANT

Consult your cases with confidence! Remember, WIMBA is your dedicated orthotic specialist, supporting your practice every step of the way.

[Submit your case](#) for medical verification easily on our website. You can book a meeting with our team to discuss your patient's needs or, if you prefer, simply fill out a form with the necessary details about your patient. We'll provide expert advice to ensure the best solution for your patient.

[CLICK TO SUBMIT YOUR CASE](#)