


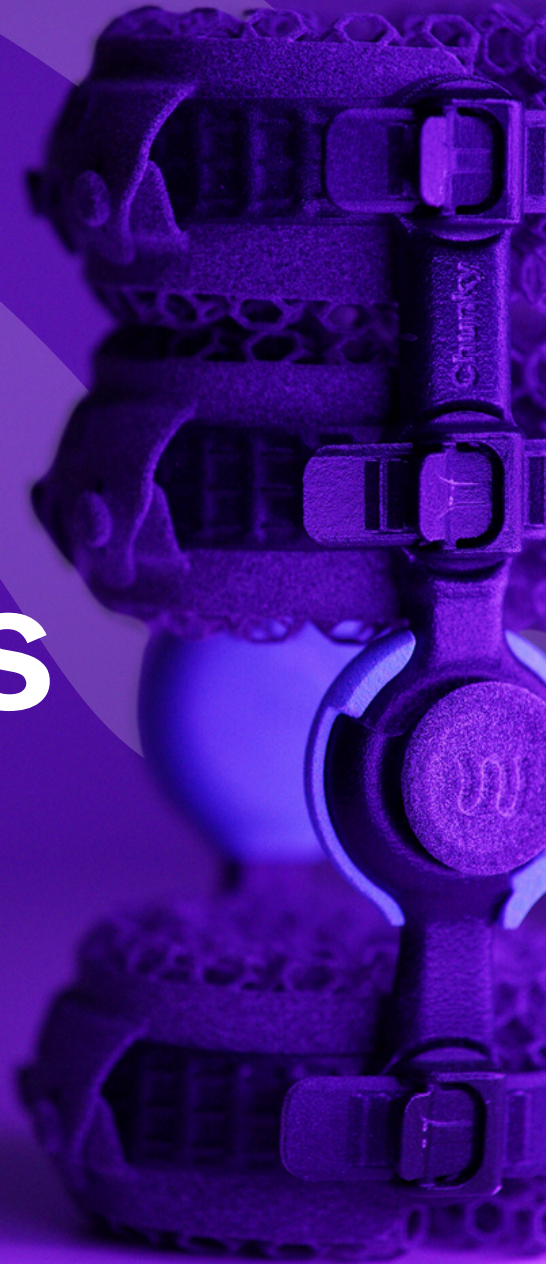

Medical Indications



Indications are suggested use only, WIMBA defers to prescribing clinician for clinical decisions.



WIMBA Carpus Orthosis:

- Carpal hyperextension (compensatory, ligament, or tendon condition)
 - mild cases
 - maximum angle in extension: 205 degrees
- Osteoarthritis in the carpal joint
- Rehabilitation Therapy (variety of uses)
- Prevention for numerous situations
- Support for obese dogs

Distal strap is placed 4.5cm from the carpal joint - we will not accept patients with smaller limbs (Minimum= 11cm limb circumference).



The number of proximal straps (1 or 2) will be based on the measurements submitted.

Red Flags

(Cases Not Suitable for Carpus Orthosis)




- Deformation of the joint
- Joint rebuilt
- Deformation of digits
- Medial or lateral instabilities

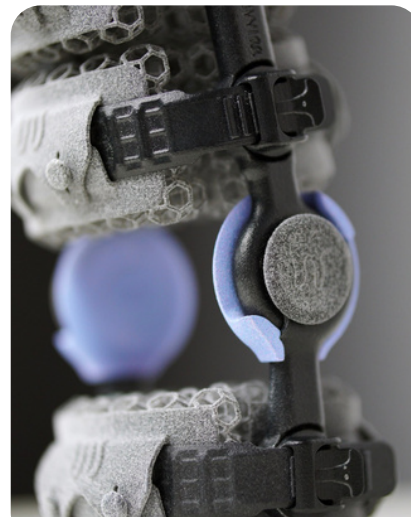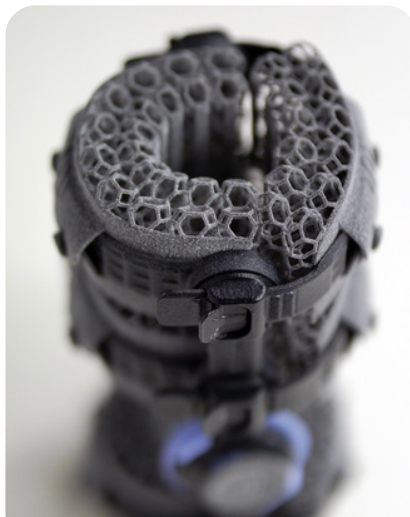

WIMBA Carpus Orthosis:



Suitable Cases

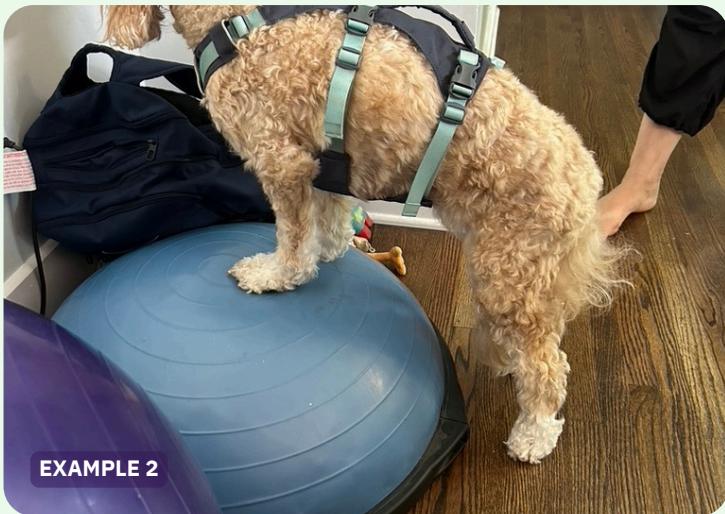

Unsuitable Cases



WIMBA Tarsus Orthosis:

- Tarsal hyperextension (compensatory, ligament, or tendon condition)
 - mild cases
 - maximum extension: 180 degrees)
- Tarsal hyperflexion (Achilles tendon/CCT injuries with no complete tear present)
 - mild cases
 - maximum flexion: 90 degrees
- Osteoarthritis in the tarsal joint
- Rehabilitation Therapy (variety of uses)
- Prevention for numerous situations
- Support for obese dogs

Distal strap is placed 4.5cm from the tarsal joint - we will not accept patients with smaller limbs at this time (Minimum= 11cm limb circumference).

The number of proximal straps (1 or 2) will be based on the measurements submitted.

Red Flags

(Cases Not Suitable for Tarsus Orthosis)





- Deformation of the joint
- Joint rebuilt
- Deformation of digits
- Medial or lateral instabilities




WIMBA Tarsus Orthosis:

✓ Suitable Cases

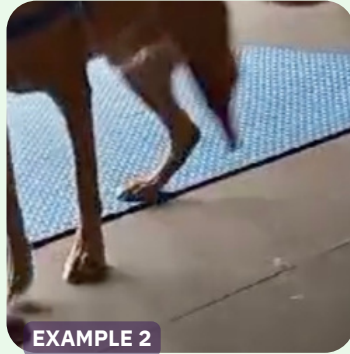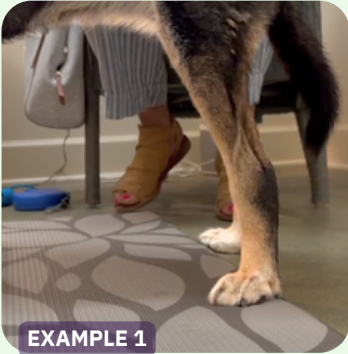

✗ Unsuitable Cases

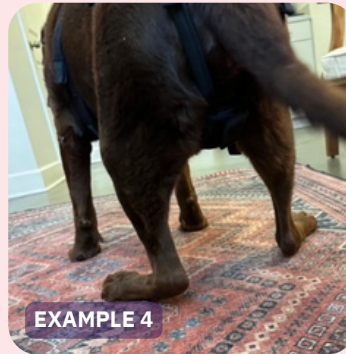


How to Order WIMBA Orthotics?


Simply download WimbaAPP available for iOS and Android devices.

Just provide a few paw measurements and two clear photos using WimbaAPP, and you'll receive a personalized orthosis that fits your patient's needs perfectly.

For 2- and 3-strap tarsus or carpus orthoses, WimbaBOX and WimbaSCAN are no longer required.

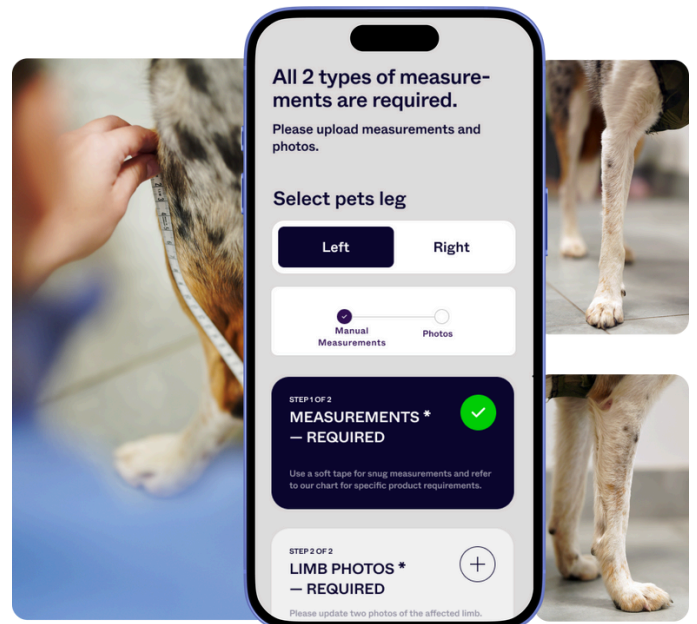

Need Help?

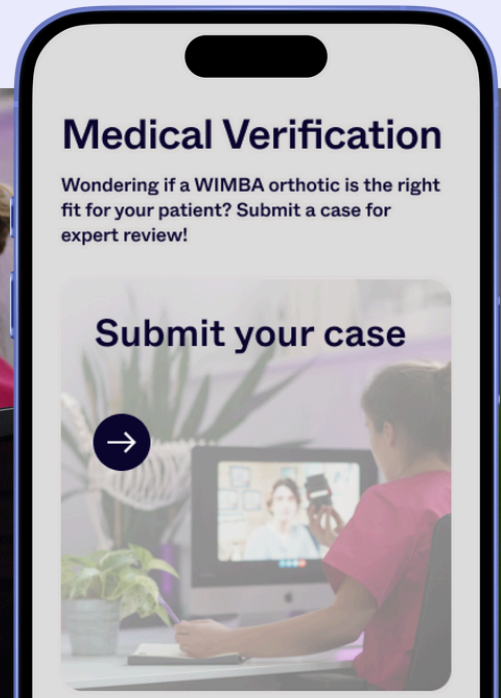

WIMBA: WE ARE YOUR ORTHOTIC CONSULTANT

Consult your cases with confidence! Remember, WIMBA is your dedicated orthotic specialist, supporting your practice every step of the way.

Submit your case for medical verification easily on our website. You can book a meeting with our team to discuss your patient's needs or, if you prefer, simply fill out a form with the necessary details about your patient. We'll provide expert advice to ensure the best solution for your patient.

[CLICK TO SUBMIT YOUR CASE](#)